Pro-Flow™ and Pro-Flow™ Plus High Performance Cannulas User's Guide

Intended Use & Features

The Pro-Tech® Pro-Flow™ and Pro-Flow™ Plus High Performance Cannulas are intended for use with pressure transducer devices during sleep disorder studies to detect respiratory airflow for recording onto a compatible physiological recorder.

These cannulas have been designed for maximum patient comfort and for optimal nasal and nasal-oral signal detection.

Caution Statements

This product is for diagnostic purposes only and is not to be used as an apnea monitor or in a life supporting or life sustaining situation.

Federal law restricts this device to sale by or on the order of a physician.


Patient Installation

Position the nasal (1) and oral (2) prongs as shown in figure 1, loop the tubing around the ears, then under the chin. Move the chin tube up under the chin to tighten. The cannula can also be secured using surgical tape. The nasal prongs of the cannula can be trimmed if desired, but must be within the nares.

The oral prong must be trimmed as close as possible to the point where the upper and lower lip come together when the mouth is closed (2). Trimming the prong slightly inside rather than slightly outside the mouth produces the best results.

Connect the cannula to the luer fitting of the pressure transducer device with a slight twist to make a secure connection.

Pro-Flow and Pro-Flow Plus High Performance Cannulas should be used with appropriate safety filters

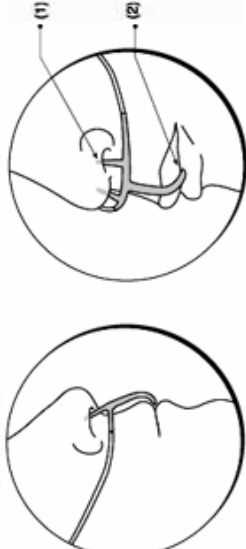Cleaning

This product was designed for single patient use only. Dispose of properly after patient use.

Warranty

Pro-Tech Services, Inc. warrants this product to be free from defects in materials and workmanship and to perform satisfactorily for one use. Should the cannula fail prematurely, the sole liability of Pro-Tech Services, Inc. is limited to repair or at its option replacement of the product with no charge for parts or labor. Under no circumstances shall Pro-Tech Services, Inc. be liable for any loss or damage, direct, consequential, or incidental, including property damage or personal injury arising from the use of, or the inability to use this product. This warranty is rendered void and Pro-Tech Services, Inc. cannot be held liable for conditions resulting from: damage, marginal performance or malfunctions caused by: misuse, abuse, neglect, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories. This warranty is in lieu of all other warranties, expressed or implied and is extended only to the original purchaser. Features and specifications are subject to change without notice.

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MT PromedT Consulting GmbH