**English**  
**CPAP Titration Kit**  
**For Portable Sleep Data Recorders**  
**(Circuit Connector with Port)**

**Intended Use & Features**  
 The CPAP Titration Kit is for use between CPAP equipment and single port sleep data recorders to provide a means to detect respiratory airflow during CPAP titration.

**Caution**  
 US Federal law restricts this device to sale by or on the order of a physician.


**Warnings**  
 The CPAP Titration Kit is provided non-sterile from the manufacturer, it should be disinfected prior to use. The filters are single use only and need to be changed between each patient use. Filters cannot be cleaned, disinfected or sterilized.

**WEEE-RoHS/Disposal**  
 If applicable, separate collection for electrical and electronic equipment per EC Directive 2002/96/EC. Otherwise, dispose of this device in accordance with local regulations.

**Specifications**

Operating Temperature	5°C (40°F) - 37°C (98°F)
Storage Temperature	-20°C (-4°F) - 60°C (140°F)
Operating/Storage Humidity	15 - 95% Non-condensing

**Installation**  
 General Guidelines:  
 Place the circuit connector in-line with the CPAP tubing near the patient mask (orientation is not critical). Attach filter on luer lok adapter, then attach the filter to the airflow input of the recorder.

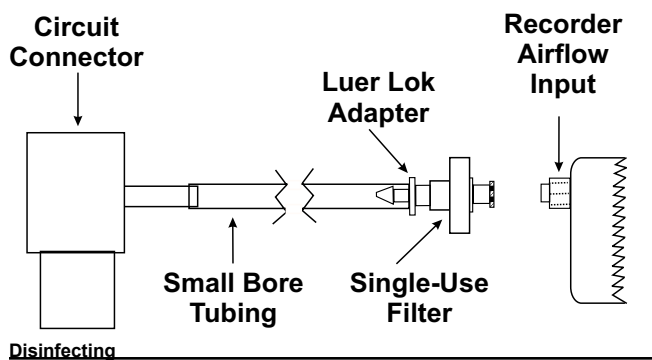**Disinfecting**  
 The CPAP Titration Kit is provided non-sterile from the manufacturer and should be disinfected prior to use – **remove the filter prior to disinfection**. The CPAP Titration Kit may be disinfected using a liquid disinfectant that is not corrosive to plastic (do not autoclave). Follow the directions provided with the liquid disinfectant. Before placing the CPAP Titration Kit back in use, allow the entire assembly to dry completely and re-attach a new safety filter.

**Warranty**  
 Pro-Tech Services, Inc. warrants this product to be free from defects in materials and workmanship for a period 30 days from the date of purchase. If the product fails to perform in accordance with this warranty, the sole liability of Pro-Tech Services, Inc. is limited to repair or at its option replacement of the product with no charge for parts or labor. Under no circumstances shall Pro-Tech Services, Inc. be liable for any indirect, special or consequential loss or damage, including economic loss, loss of business or overhead, or any punitive damages, arising from the use of, or the inability to use this product. This warranty is rendered void and Pro-Tech Services, Inc. shall not be liable for conditions resulting from damage, marginal performance or malfunctions caused by misuse, abuse, neglect, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories, or any other defect not related to materials or workmanship. This warranty is in lieu of all other warranties, expressed or implied and is extended only to the original purchaser. Features and specifications are subject to change without notice.

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EC REP Advena Ltd. Hereford HR4 9DQ UK.