<b>Dual Periodic Limb Movement Sensor</b>
<i>English</i>
<b>INTENDED USE &amp; FEATURES</b>

The Dual PLM Sensor is intended for use during sleep disorder studies to detect periodic limb movements for recording onto a physiological recorder.

#### CAUTION STATEMENTS

US Federal law restricts this device to sale by or on the order of a physician.

#### WEEE-RoHS/DISPOSAL


☒ If applicable, separate collection for electrical and electronic equipment per EC Directive 2002/96/EC. Otherwise, dispose of this device in accordance with local regulations.

#### SPECIFICATIONS

Operating Temperature	5°C (40°F) - 40°C (104°F)
Storage Temperature	-20°C (-4°F) - 60°C (140°F)
Operating/Storage Humidity	15 - 95% Non-condensing

#### SENSOR INSTALLATION (PATIENT SETUP)

Select an appropriately sized strap (3). Place the PLM Sensor element into the sensor pocket (4). Place the strap assembly around the patient's ankle or wrist as shown. The strap should be oriented so that the smooth side of the strap is in contact with the patient and the looped/napped side is facing out. The strap is then secured to itself by passing the end through the rectangular loop located on one end of the strap and then applying the hooked section of the strap to the fuzzy section. The hooked section will also stick to the smooth side of the strap, allowing the strap to be made much smaller by wrapping it back on itself. Straps should be slightly snug, but not to the point of patient discomfort. Run the lead wires up the patient's legs or arms to the recorder. Tape the wire leads as shown (1 & 2) to provide strain relief to the sensors. Straps may also be used further up the patient's legs or arms to prevent the wires from being pulled on.

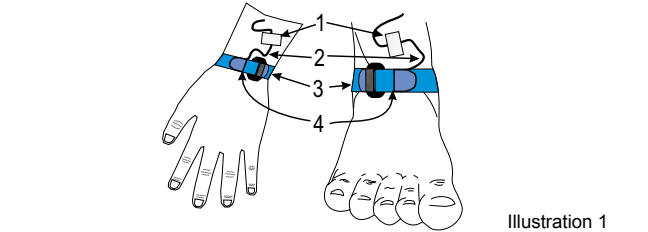- Medical Tape Strain Relief
- Lead Wire Stress Loop
- PLM Strap
- PLM Sensor Pocket

#### INSTRUMENT CONNECTIONS

Connect the safety connector to the appropriate input jack on the recorder. CAUTION: Grasp the connector when plugging and unplugging the Sensor from the recorder or polygraph jack box. Damage caused by pulling on the wires is not covered under warranty and is not repairable.

#### SAMPLE RECORDER TRACINGS

#### CLEANING

Wipe clean with a hospital approved disinfectant that is not corrosive to plastic or metal.

Do not immerse the sensor in cleaning solution. Immersing the sensors will damage the device and voids the product warranty.

The straps may be washed in soap and water then air dried.

#### WARRANTY

Pro-Tech Services, Inc. warrants this product to be free from defects in materials and workmanship for a period of six months from the date of sale by Pro-Tech Services, Inc. If the product fails to perform in accordance with this warranty, the sole liability of Pro-Tech Services, Inc. is limited to repair or at its option replacement of the product with no charge for parts or labor. Under no circumstances shall Pro-Tech Services, Inc. be liable for any indirect, special or consequential loss or damage, including economic loss, loss of business or overhead, or any punitive damages, arising from the use of, or the inability to use this product. This warranty is rendered void and Pro-Tech Services, Inc shall not be liable for conditions resulting from damage, marginal performance or malfunctions caused by misuse, abuse, neglect, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories, or any other defect not related to materials or workmanship. This warranty is in lieu of all other warranties, express or implied, and is extended only to the original purchaser. Features and specifications are subject to change without notice.

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