## English Snore Sensor

### Intended Use

The snore sensor is intended for use during sleep disorder studies to detect tracheal sounds for recording onto a physiological recorder. The sensor plugs directly into an AC coupled electrode jackbox, no batteries or interface box is required.

### Caution Statements

US Federal law restricts this device to sale by or on the order of a physician.

### WEEE-RoHS/Disposal

♻ If applicable, separate collection for electrical and electronic equipment per EC Directive 2002/96/EC. Otherwise, dispose of this device in accordance with local regulations.

### Specifications

Operating Temperature	5°C (40°F) - 40°C (104°F)
Storage Temperature	-20°C (-4°F) - 60°C (140°F)
Operating/Storage Humidity	15 - 95% Non-condensing

### Sensor Installation

The snore sensor is installed onto the patient as shown in the illustration below. Position the sensor onto the neck (1) in an area that gives maximum signal on the recording device. The sensor and lead wire (4) should be carefully secured in position using surgical tape (3).

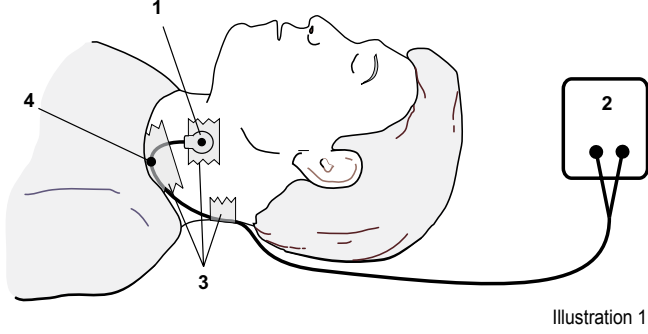
Illustration 1

### Instrument Connections

Connect the safety connectors of the snore sensor to the desired inputs of a recorder (2). CAUTION: Grass safety plugs firmly when unplugging the snore sensor from the jack box. Damage caused by jerking on the wires is not covered under the warranty and is not repairable.

### Instrument Settings

**Sensitivity** - 2 - 75µV/mm. Adjustment of the sensitivity up or down is typically required.

**Low Frequency Filter / Time Constant** - 5 Hz / 0.03 seconds

**High Frequency Filter** - 70 Hz.

### Sample Recorder Tracing

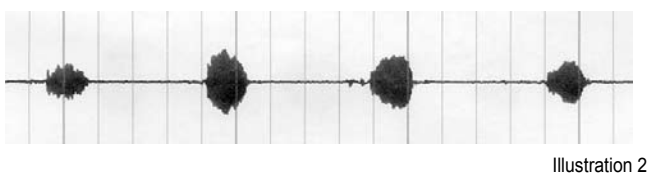
Illustration 2

### Cleaning

The snore sensor may be wiped clean with a hospital approved disinfectant. CAUTION: The snore sensor is not liquid tight, do not soak the sensor in cleaning solution.

### Warranty

Pro-Tech Services, Inc. warrants this product to be free from defects in materials and workmanship for a period of six months from the date of sale by Pro-Tech Services, Inc. If the product fails to perform in accordance with this warranty, the sole liability of Pro-Tech Services, Inc. is limited to repair or at its option replacement of the product with no charge for parts or labor. Under no circumstances shall Pro-Tech Services, Inc. be liable for any indirect, special or consequential loss or damage, including economic loss, loss of business or overhead, or any punitive damages, arising from the use of, or the inability to use this product. This warranty is rendered void and Pro-Tech Services, Inc. shall not be liable for conditions resulting from damage, marginal performance or malfunctions caused by misuse, abuse, neglect, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories, or any other defect not related to materials or workmanship. This warranty is in lieu of all other warranties, express or implied, and is extended only to the original purchaser. Features and specifications are subject to change without notice.

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All rights reserved.